


Grab Rail Care & Maintenance

Thank you for choosing a Goldfern Mobility Grab Rail! Regular care and maintenance will extend the life of your product, as well as keeping it clean and safe to use. By routinely inspecting and cleaning your Grab Rail, you can prevent potential accidents and create a safer, more secure environment.

- Wipe your Grab Rail with a soft, damp cloth for daily cleaning.
- **Do not** use a hard or abrasive cleaning tool to clean your Grab Rail. Tools such as steel wire balls, nylon brushes or other objects can scratch and deteriorate the surface of your product.

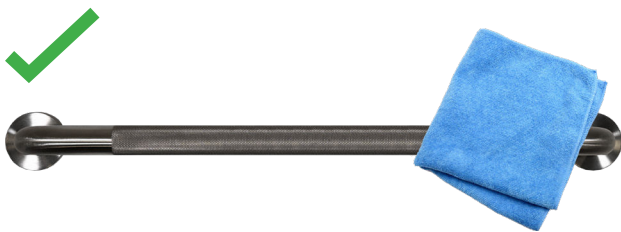

- When the Grab Rail comes into contact with water, wipe with a soft cloth immediately after use to prevent any water damage or stains.
- Every one or two weeks, or when the Grab Rail becomes visibly dirty, clean with a neutral detergent or other regular surface cleaner, then thoroughly wash with water. Dry immediately with a soft cloth to prevent water damage.
- **Do not** use highly corrosive/harsh cleaning solutions on your Grab Rail. They can cause surface damage, and even if not visible immediately, the material can change colour and deform over time.